# Release Notes

## coDiagnostiX 10.3

### English

#### New license option: coDiagnostiX EASY PRINT

For clinicians who own a 3D printer and collaborate with a planning service, coDiagnostiX 10.3 now features a new license option: EASY PRINT. This license option allows you to receive finalized treatment plans via caseXchange from your planning service for printing surgical guides on your own local 3D printer.\*

#### Model scan import

The user interface of the model scan import has been improved with a new dynamic tool panel, toolbar and work view for better ease of use. Among others, new and improved repair, detection and cutting functionalities have also been added.

#### Other

- The abutment dialog can now show all implants and abutments from the treatment plan. This makes it easier to adjust the abutment rotation.
- Deleting treatment plans was moved from the plan menu to the plan management dialog. This should help to prevent users from accidentally deleting treatment plans.
- An issue regarding proxy servers requiring authentication has been fixed.

#### Sleeves

- CAMLOG Progressive Line Sleeves
- Medentika Procone

\*Available through Straumann Smile in a Box starting from 1 January 2021, and for all validated laboratories starting from 1 March 2021.

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