

# Release Notes 10.4 & 9.17

We're constantly working to improve your experience with coDiagnostiX. Below you will find a summary of the items fixed or improved upon in this update.

## NEW SLEEVE SYSTEMS

New sleeve systems for Anthogyr, Neodent, Bredent and Thommen are available.

## SURGICAL GUIDE DESIGN IMPROVEMENTS

The following improvements for surgical guide designs sent to Straumann production centers were made to optimize speed and accuracy of the production process.

**Guide type:** Users of coDiagnostiX 10.4 are now asked whether their guide is intended for cutting bone.

**Accessory selection:** When placing their order, users can now choose between the three options "No Accessories", "Sleeves only" or "Sleeves and pins" depending on their current needs.

**Customer number:** users are now asked to enter their customer number when placing their order to ensure the order is correctly assigned to its intended recipient.

✕
Send Order

**Send order**

Compose the order you would like to send.

?

**Straumann customer number**

123456

↻ Change...

**Shipping address**

**John Doe, Dental Wings GmbH, Düsseldorfer Platz 1, 09111 Chemnitz Germany**

↻ Select shipping address... ↻ Reset

**Accessories**

Add further items to your order

Pins and drill sleeves ▼

**Surgical guide usage**

Please mark all guides that you use for the guided sawing of the bone in the list below.

Surgical guide	Sleeves	Cut profile	Bone support region	Modification
<input type="checkbox"/> Surgical guide	1	No	No	No
<input checked="" type="checkbox"/> Surgical guide 1	0	Yes	No	No

< Back
Send
Cancel

## CALIBRATION MATRIX PRINTING

We have made enhancements to the process of printing a calibration matrix. Users can now select which printer they are using, to ensure that the calibration matrix is generated in a way which is optimized for that printer, for example when using a RapidShape printer, or other DLP printers.