

## 10.5 & 9.18 Release Notes


### Improvements delivered with this update

#### Import of Xorder files

With the new update, coDiagnostiX will be able to import Xorder files through the Scan Import dialog box directly from a storage location. This will be an alternative option to importing such files from DWOS Connect, which will further improve the software's connectivity to scanner devices.


#### Switch between implant and abutment interface

The user will now be able to select between implant and abutment interface when exporting the patient case through the Virtual Planning Export. This improvement, which will be available for the Xorder export in a first step, will better tailor the export to the subsequent CAD workflow, making it possible to export the patient case without the abutment if the implant interface option is selected.

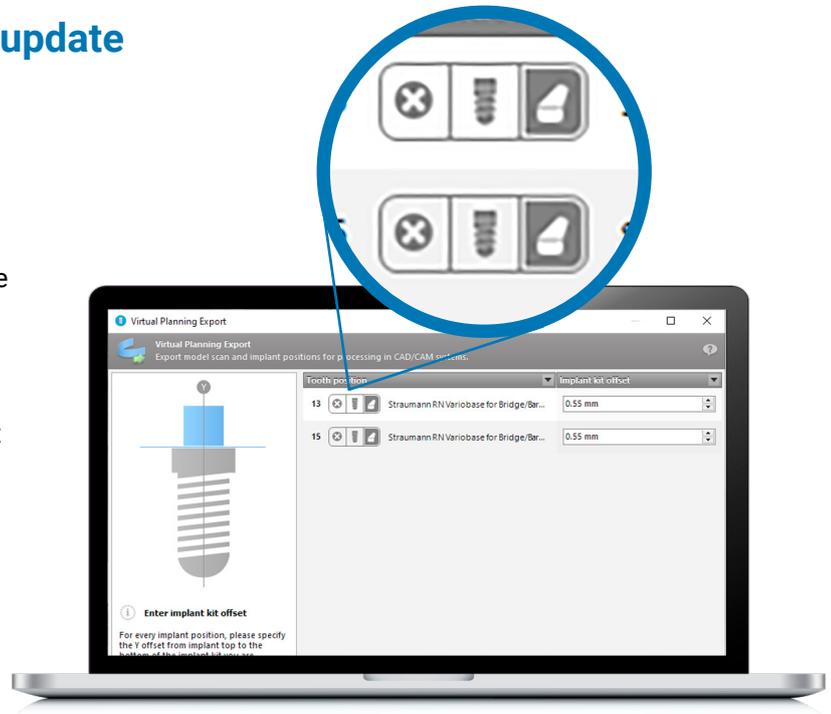

#### Mouse thumb wheel function

coDiagnostiX now offers the option to adjust the viewing perspective in the 3D, Panoramic and Tangential views with the help of the thumb wheel of the mouse. Scrolling the wheel will rotate the views and helps to further speed up work.

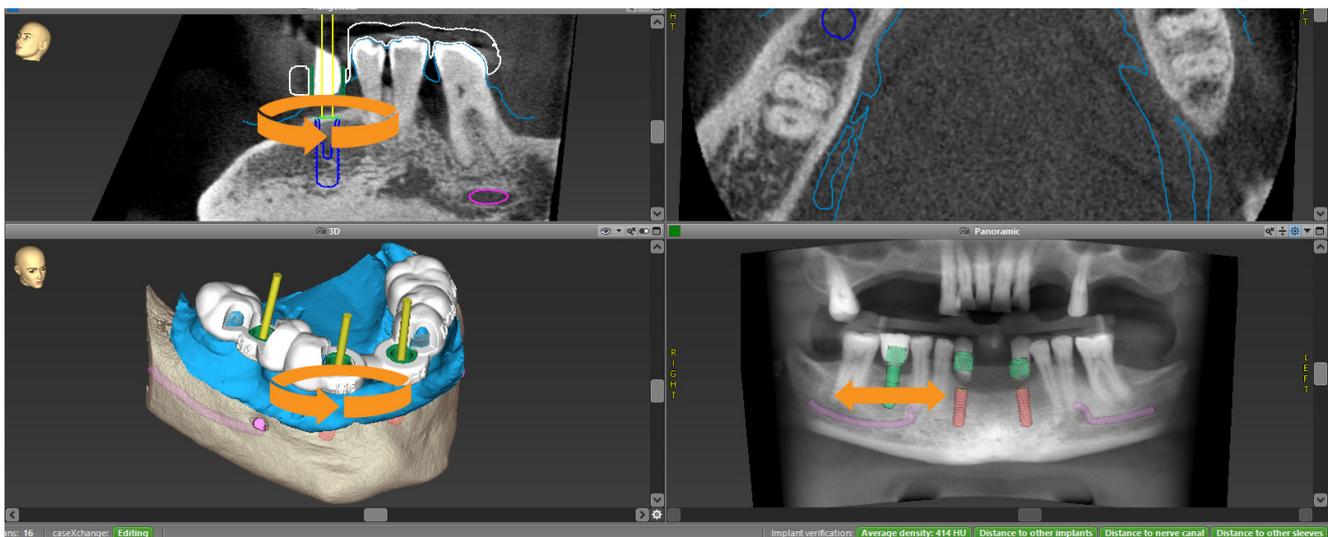

#### Integration of additional sleeves

This update adds an additional BioHorizon sleeve as well as Camlog pilot sleeves to the coDiagnostiX database.