

coDiagnostiX 10.9.1

Improvements

- Added support for the NUVO sleeve system.
- Added support for the Neodent Zi 5.0 sleeve.
- Added support for Straumann metal scan bodies to Treatment Evaluation.
- The about box now shows three numbers of the version number instead of two (10.9.1 instead of 10.9).
- The about box now displays the UDI PI (production identifier).
- The about box now provides a list of country specific production labels.

coDiagnostiX 10.9

Improvements

- The Straumann surgical protocols were all switched from 4 to 3 bone classes.
- The Straumann TLX/BLX and TLC/BLC surgical protocols now both follow the from-pilot-to-final approach.
- Fixed an issue in the Neodent EasyGuide surgical protocol where columns were too narrow and cropped the contained textual information.
- Fixed an issue where the abutment angulation measurement dialog changed the selected implant in the planning screen.
- Fixed an issue where surgical protocol information was not correctly displayed when using Medentika MedentiGuide.
- Fixed a frequent crash that occurred when using the SQL Network Database feature.
- Fixed a crash that occurred when closing the software without a dongle attached.