

coDiagnostiX 10.9.2

AXS license update

To use coDiagnostiX after the 5th of August, install this update if you are using a subscription license without a hardware dongle.

Straumann iGuide sleeve system

This update adds support for the new iGuide system. Make sure also to install the accompanying MasterDB update.

Support for new Straumann Falcon workflows

coDiagnostiX now supports workflows using three fixed landmarks and three natural landmarks. See the Falcon user manual for more information.

Other

Small functionality and stability fixes.

coDiagnostiX 10.9.1

Improvements

- Added support for the NUVO sleeve system.
- Added support for the Neodent Zi 5.0 sleeve.
- Added support for Straumann metal scan bodies to Treatment Evaluation.
- The about box now shows three numbers of the version number instead of two (10.9.1 instead of 10.9).
- The about box now displays the complete UDI, including UDI-DI (device identifier) and UDI-PI (production identifier).
- The about box now provides a list of country-specific labels.

coDiagnostiX 10.9

Improvements

- The Straumann surgical protocols were all switched from 4 to 3 bone classes.
- The Straumann TLX/BLX and TLC/BLC surgical protocols now both follow the from-pilot-to-final approach.
- Fixed an issue in the Neodent EasyGuide surgical protocol where columns were too narrow and cropped the contained textual information.
- Fixed an issue where the abutment angulation measurement dialog changed the selected implant in the planning screen.
- Fixed an issue where surgical protocol information was not correctly displayed when using Medentika MedentiGuide.
- Fixed a frequent crash that occurred when using the SQL Network Database feature.
- Fixed a crash that occurred when closing the software without a dongle attached.