Release Notes

coDiagnostiX 9.10

English

coDiagnostiX EASY

The coDiagnostiX EASY step-by-step implant planning wizard has been completely reworked. It is easy to learn and use yet covers a wide range of treatment scenarios. It offers the full workflow from dataset alignment to export of surgical guides.

Surgical Guide Designer

3D objects can now be added to or subtracted from the guide. This enables adding e.g. a digital prosthetic setup or a scanned denture to the guide, in order to register it to the bite during surgery.

Interface to Mininavident DENACAM Implant Surgery

With coDiagnostiX 9.10, your planning data can be exported to use within DENACAM Implant Surgery, a 3D navigation system for implantology.

New sleeve systems

- Biomet 3i Tapered Navigator
- TBR GUIDED CLASSIC/MCI
- Titaniumfix
- Extension for Neodent GM

Surgical protocols

- Added support for Straumann BLT 2.9.
- Improved readability by changing the way of displaying color codes.

Further improvements

- coDiagnostiX now supports the Polygon File Format (PLY) as an importable file format. It offers support for e.g. colored intra-oral scans.
- The DICOM import now supports the ISO 2022 IR 13 string format to improve compatibility with Japanese characters.

- In views and on printouts, the readability of measurements was improved by making the font size adjustable.
- A button to quickly add measurements can be added to the toolbar.
- A button to quickly switch view configurations can be added to the toolbar.
- The tooth position of implants can now be displayed in the views.
- New option to disable caseXchange and Order Management notifications.
- Performance optimizations

Known Issues

- coPeriodontiX is temporarily not accessible in version 9.10.

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