Release Notes

coDiagnostiX 9.14

English

New features

- The loading speed of the implant selection dialog has been significantly improved.
- When a lateral pin is added, it is now automatically placed at the selected tooth and at an angle. This ensures that the pin is in the best position to start fine alignment. In addition, pins are now assigned an undefined tooth position (XX) to prevent them from blocking the tooth position for implants.
- A new printout has been added to allow users more customization options. It contains all images stored in the Image Management for a particular patient.

Improvements in EASY mode

- It's now possible to measure the distances in the Implant Alignment step in EASY mode. Previously, this function was available only in EXPERT mode.
- The Dataset Preparation step in EASY mode now features a segmentation threshold slider. It allows users to optimize the appearance of their 3D reconstruction from scan data more easily.

Bugfixing

- Increased stability of the surgical guide designer.
- Fixed a rare problem where producers were not able to read datasets from their locked clients.
- Fixed a rare problem that occurred in the Straumann surgical protocol print-out. Sometimes, the order of tooth positions did not correctly match the dental notation system that was chosen.
- Fixed a problem with five CAMLOG implants, where the visible diameter of the 3D object was 0.4 mm smaller than the value from the catalog. The following implants were affected: K1054.4311, K1054.4313, K1054.4316, K1053.4316, and C1086.3809.

Known issues

- coPeriodontiX is not accessible until further notice.

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