Release Notes

coDiagnostiX 9.15

English

3Shape TRIOS integration

- coDiagnostiX is now fully integrated with 3Shape's TRIOS scanner! Users can send IO scans from the doctor's TRIOS directly to the lab's coDiagnostiX installation via 3Shape Communicate or to their own coDiagnostiX installation via DWOS Gateway.

Integrated Training

- With the new integrated training, coDiagnostiX offers an additional training option and how-to instructions to newcomers and learners alike, visually guiding users step-by-step. Lessons are constantly expanded and will range from basic functions such as DICOM import and dataset creation to advanced functions such as segmentation.

New sleeve systems

- New sleeve systems from Medentika, Anthogyr, Zimmer and Straumann South Korea have been added.

Angulated abutment platform of implants improved

- The planning process for implants with an angulated abutment platform has been improved: angulated zygomatic implants can now be fitted with abutments and it is possible to adjust the rotation of profiled implants according to the bone profile.

Bug fixes

- A problem with the matching of surface scans imported during a Synergy connection has been fixed.
- Fixed a bug that occasionally prevented paper printouts after a computer reboot.
- Fixed a bug in the implant selection dialog that left tooth positions unmarked even though an implant was planned in that position.

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