Release Notes

coDiagnostiX 9.16

English

New license option: coDiagnostiX EASY PRINT

For clinicians who own a 3D printer and collaborate with a planning service, coDiagnostiX 9.16 now features a new license option: EASY PRINT. This license option allows you to receive finalized treatment plans via caseXchange from your planning service for printing surgical guides on your own local 3D printer.*

Model scan import

We improved the handling of imported surface scans and other 3D objects which are not optimal for treatment planning to better aid virtual tooth extraction and mesh optimization. This new functionality can be found in the Model Scan Import dialog and in the context menu of imported model scans and other 3D objects.

Other

- The abutment dialog can now show all implants and abutments from the treatment plan. This makes it easier to adjust the abutment rotation.
- Deleting treatment plans was moved from the plan menu to the plan management dialog. This should help to prevent users from accidentally deleting treatment plans.
- An issue regarding proxy servers requiring authentication has been fixed.

Sleeves

- CAMLOG Progressive Line Sleeves
- Medentika Procone

*Available through Straumann Smile in a Box starting from 1 January 2021, and for all validated laboratories starting from 1 March 2021.

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