Release Notes 9.5.2

English

New features and improvements within coDiagnostiX™ 9.5

Bone supported drill guides

coDiagnostiX™ 9.5 allows to create bone-supported drill guides. It is also possible to design drill guides with a combination of teeth and bone support. For this purpose a new optional wizard page was added to the digital drill guide wizard.

Further drill guide improvements

- In addition to the sleeve socket diameter, the height of the socket can now be specified.
- For selected implant series, you can now have rotation markers engraved into the drill guide to enable precise placement of previously planned restoration.

DWOS Synergy

Integrated real-time implant and restoration design data.

DWOS Synergy opens up a completely seamless and time-saving workflow between coDiagnostiX™ and DWOS. Transfer your coDiagnostiX™ implant planning to DWOS and get back the restoration planning in real time. Collaborate in your local network with just one click - or over the Internet with a free DWOS Synergy account.

Virtual Planning Export

Our Virtual Planning Export has been completely redesigned and contains the following new features:

- Virtual model builder
- Multi STL file export
- Selectable export coordinate systems (STL)
- Selection of scanbodies from various manufacturers
- Better user interface

Further improvements of coDiagnostiX™ 9.5

- Surgical protocol for Bredent implants
- New sleeve systems: Dental Wings, Neodent, Implant Solutions
- Inline comment function for planning objects
- Filter implants by geographical region
- Improved workflow for SQL Network Database
- Improved performance and stability
- Improved quality of surface meshes exported with Virtual Planning Export (#3974, #4005).

- Solved problems in Synergy, Digital Drill Guide wizard, DICOM import, coDiagnostiX EASY, and printouts.

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