Release Notes 9.6

English

DWOS Synergy now supports the transfer of stock abutments.

Within a DWOS Synergy session, abutments planned in coDiagnostiX and contained in the coDiagnostiX database will now be transferred to DWOS. This allows for easier planning and design of all-on-four cases and restorations.

Transfer of planning data to bone block augmentation manufacturers

coDiagnostiX patient cases with bone and implant planning information can now be exported or directly sent (via caseXchange) to bone block manufacturers to have them produce customized bone blocks.

Transfer of planning data to radiographic assessment service providers

coDiagnostiX now allows patient cases to be sent via caseXchange to service providers to obtain radiographic assessment.

Transfer of planning data to Bien-Air's iChiropro app

coDiagnostiX now offers an interface to iChiropro which provides surgical protocol information and pre-programmed operating sequences.

Support for new sleeve systems added

- The database now includes 3D Diagnostix and Neoss sleeve systems.
- Improved support for Bego's sleeve system. coDiagnostiX now supports non rotation-symmetric sleeve bodies.

Several user interface improvements in caseXchange, e.g.

- Quick search in caseXchange
- Plan name displayed in order management
- Email address clickable in order management
- Contact information can be copied to clipboard
- Hotkeys to access caseXchange and order management from start screen

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