Release Notes 9.7

English

DWOS Synergy Improvements

Complete prosthetic information

All prosthetic elements from DWOS are now transferred to coDiagnostiX. This includes bridge elements, pontics, crowns etc.

In turn, bone information can be sent from coDiagnostiX to DWOS.

Additional information when sharing cases

When sharing cases through DWOS Synergy, the name of the laboratory/clinician is now transferred and displayed for easier identification.

Improvements in drill guide design

Dual scan method supported

coDiagnostiX now supports the direct import of a segmentation from a second dataset.

A special option allows to use a denture's intaglio surface for drill guide design.

Lateral fixation improved

Support for drill guides with lateral fixation has been improved. Lateral pins can now be assigned to an undefined tooth position and are automatically included in the drill guide.

Regular tooth positions are no longer occupied by fixation pins.

Free design of support surfaces

The new drawing tool lets the user freely design a drill guide's support surface to realize e.g. palatal or gingiva support.

Palatal support may provide a more reliable fit of drill guides in edentulous cases.

Connector preview

A new option in the drill guide design wizard displays the sequence of the guide's contact points indicating positions where connectors will be generated.

Presets for labels

coDiagnostiX now offers factory and user-defined presets for drill guide labels to speed up work.

Easier export of sleeve calibration matrix

The calibration matrix for a sleeve can now be exported directly through a sleeve's context menu and is, thus, available prior to drill guide design.

Improved support for edentulous cases

- Dual scan method supported
- Improved support for lateral fixation
- Free design of supporting surface to realize palatal support
- Display of abutment axes for angulated abutments (useful e.g. for implant-borne full dentures)

Further improvements

Sleeve systems and selection box

- With Dentaurem and Sweden & Martina, two new sleeve systems have been added.
- The sequence of sleeve systems displayed in the selection box has been reworked. Sleeve systems offered by the implant manufacturer are now shown on top.

Improved iChiropro support

- Additional implant manufacturers are supported.
- More information are transferred to iChiropro, including:
 - the Implant Details printout from coDiagnostiX
 - surgical protocol information for Straumann implants

Support & information center

- The new support and information center provides centralized access for all help, support and training issues.
- A news section informs the user about pending updates and upcoming events.

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