Release Notes 9.7.5

English

- coDiagnostiX does now offer information on whether a dataset has been unlocked for production, when this was done and by which license number. This information can be found in the Plan menu under “Check lock state for production”.
- In Synergy, the doctor's name which is entered in DWOS/CARES is now displayed in the inbox to make it easier to identify the case if several cases have been shared.
- For digital drill guides, the diameter and height of inspection windows can now be set down to 0.4 millimeters. This allows advanced guide features like flushing channels (small diameter) or slots for drill handles (low height).
- When a dataset is locked after archiving, it can now be unlocked even if the corresponding message had been dismissed. In the main menu, go to Patient > Remove archive lock.
- Added support for the new Straumann 2.2 mm sleeve.
- Dental Wings sleeve system renamed as Meisinger.
- Revised the surgical protocol for Straumann 2.8 mm sleeves and 8 mm BLT implants.
- Revised 3DDX and Neoss sleeve systems.

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