Release Notes 9.8

English

Improved support for edentulous

Automatic placement of correctly angulated screw-retained abutments based on the implants' position and angulation adds further support for screw-retained bars and bridges via DWOS Synergy and enhances existing edentulous workflows.

DWOS Connect interface

Dentists who own a Dental Wings scanner can now seamlessly transfer their scans via DWOS Connect to create perfectly fitting drill guides in coDiagnostiX.

Native DICOM data stored

With coDiagnostiX 9.8, the full DICOM grayscale range is stored with the dataset abolishing the necessity to adjust presets during import. This further simplifies DICOM import and allows the user to adjust the grayscale range and modify the contrast any time during their planning activities to improve visibility.

caseXchange improvements

- Comments function: Add supplementing information when transmitting data.
- Transmission Status information: Provides feedback to the sender when the dataset has been downloaded by the recipient.
- Time-saving Send Back function: Quickly and easily return a commented or approved case planning to the sender.
- Tidy-up function: Hides completed cases on the transfer list making it clearer and easier to handle.
- One-Click Download: Reduces the number of clicks to speed up work.
- Download Again function: Repeated download of a dataset with one click.

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